- Distributors Germany / Vertriebe Inland
- Distributors Europe / Vertriebe Europa
- Distributors Germany & abroad worldwide /
Vertriebe Inland & Ausland weltweit

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2024-06-26

Subjects:

1. **Field Safety Notes (FSN) of the electrode manufacturer – Leonhard Lang GmbH**

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1. Field Safety Notes (FSN) of the electrode manufacturer – Leonhard Lang GmbH

We would like to inform you that a FSN (Field Safety Notice) has been issued by the manufacturer of our corPatch easy therapy electrodes, Leonhard Lang GmbH.

The affected therapy electrode types listed below (production date before 11/2022) will **not be actively** replaced. The user will be informed via the FSN about the correct troubleshooting in the occurrence event. Only conspicuous therapy electrodes will be replaced by GS Elektromedizinische Geräte G. Stemple GmbH. Newer therapy electrodes are not affected.

REF 04324.3 corPatch easy

REF 05120.1 corPatch easy pre-connected

REF 05120.3 corPatch easy pediatric

If an affected therapy electrode is used, in very rare cases the message "Connect electrodes" may appear in defibrillator mode, despite the therapy electrodes being connected and therapy is therefore not possible.

If the message "Connect therapy electrodes" appears after connecting the electrode to the defibrillator, use a replacement electrode immediately.

All affected sales and service partners will receive separate information from Leonard Lang GmbH shortly.

If you do not belong to the affected group of people and have not received the FSN, but you are still experiencing the problem, please contact GS Elektromedizinische Geräte G. Stemple GmbH, service@corpuls.com immediately.

We apologize for any inconvenience caused and thank you for your support.

In case of questions do not hesitate to contact our Customer Support Dept., Mr. Daniel Rampp, Daniel.Rampp@corpuls.com.

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REF 04324.3 corPatch easy
REF 05120.1 corPatch easy pre-connected
REF 05120.3 corPatch easy pediatric

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1. **Sicherheitshinweis (FSN) des Elektrodenherstellers – Leonhard Lang GmbH**