

IMPORTANT PRODUCT NOTICE
ROTEM® *sigma* ROTROL P Part No. 555202

August 14th, 2025

Dear Valued ROTEM Customer:

This notification is intended to advise your facility regarding a potential issue identified with ROTEM *sigma* ROTROL P lot 32400122.

The table below identifies the affected product:

Product Name	Part No.	Lot No.	Expiration Date (YYYY-MM-DD)	UDI-DI
ROTEM <i>sigma</i> ROTROL P	555202	32400122	2026-01-31	04260160470358

- **Issue Description and Impact**

- Werfen has identified that the above-mentioned lot of ROTEM *sigma* ROTROL P has a higher than expected likelihood of generating quality control out of range results in INTEM C CT or HEPTTEM C CT assays in conjunction with cartridge lots numbers between S240101 and S250209.

- **Risks to Health:**

- In the setting where Rotrol P fails QC due to assigned values the system will be rendered unusable. Hence, the treating clinician will likely resort to clinical decision-making for bleeding management and transfusion decisions based on clinical expertise; other available laboratory and diagnostic results; real-time, specific patient considerations; and institution-specific transfusion guidelines/protocols. A subset of these clinical decisions may lead to transfusion decisions (packed red blood cells, fresh frozen plasma or equivalent, cryoprecipitate or equivalent, or platelets) that may differ from viscoelastic-guided driven decisions. A corrective action has been opened to identify mitigations to prevent this problem from reoccurring in the future.

- **Mandatory Customer Actions**

Based on the above, please take the following immediate actions:

- **If** you are using the affected ROTEM *sigma* ROTROL P lot number identified above in your facility, please utilize the target range sheet received with this information for all ROTEM *sigma* cartridge complete and complete plus hep LOTs S240101 and higher for quality control measurements.
- **Document** the acknowledgement on the Customer Reply Form and **return** the completed and signed form to the fax number or e-mail address listed.
- **Share** this information with your clinical users and laboratory staff and follow your internal procedures.
- **Forward** this notification to all affected locations within your facility.
- **Report** all device-related suspected serious incidents to the manufacturer, distributor, local contact point, and, if appropriate, the National Competent Authority.
- **Retain** a copy of this notification for your records.

- **Customer Reply Form**

The customer reply form can be communicated to Werfen via the below options:

- e-mail address: tem-ra@werfen.com
- Fax no.: + 49-89-45429522

- **Contact information for questions**

- For technical questions please contact your local Werfen representative

	Customer Letter	<i>Doc#:</i> <SAP Doc#> <i>Rev:</i> <00> <i>Page:</i> 2 von 1 of 2
---	------------------------	--

We appreciate your prompt attention to this Important Product Notice.

Sincerely,

Signed by David Jacob
 *David Jacob* | I approve this document
2025-08-14 | 1:43:35 PM CEST
680C197BCA4643899FB4C4DDA06F4347

David Jacob
Director of Quality Assurance and Regulatory Affairs, PBM PRRC
Tem Innovations GmbH
SRN: DE-MF-000012176